The ProReveal Process Inspection Mode



Select a surgical instrument which has been through the wash process.



Place the surgical instrument onto the paper.



Spray the surgical instrument with the ProReveal spray, from a distance of 30cm.



Close the drawer to continue the automatic sequence; a timer on the screen shows time to completion.

Click the Start button on

the ProReveal software,


then follow instructions on the screen to test the

surgical instrument




ProReveal displays the amount of any residual protein left on the instrument. Eg:

10 μg

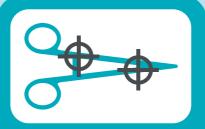

The ProReveal software displays a detailed image of the surgical instrument under test, highlighting the exact location of any detected contamination which might remain after the washing process.

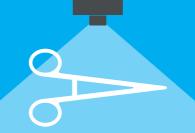

The ProReveal software waits for the reagent to 'develop', then uses fluorescence image processing to detect and measure tiny quantities of contamination.



A full report showing image and test data is available for printing out or to be kept in a logbook or other departmental record.

The ProReveal viewer ejects the drawer automatically on completion of the test.

The surgical instrument, having been sprayed and tested, must now be returned to the wash process.

S Y N O P T I C S







www.synopticshealth.com